



# Neogen ELISA Drug Detection

## Forensic Drug Panels

### Impaired HealthCare Professional

- Benzodiazepines (Oxazepam/Clonazepam)
- Buprenorphine
- Buprenorphine/Norbuprenorphine
- Butorphanol
- Carisoprodol
- Fentanyl
- Ketamine
- Meperidine
- Methadone
- Methadone/LAAM
- Methylphenidate
- Naltrexone/Nalbuphine
- Oxycodone/Oxymorphone
- Pentazocine
- Propoxyphene

### Drug Facilitated Sexual Assault

- Amphetamine
- Amphetamine Specific
- Benzodiazepines (Oxazepam/Clonazepam)
- Barbiturates
- Carisoprodol
- Citalopram
- Clonidine
- Cocaine/BZE
- Dextromethorphan
- Fentanyl
- Flunitrazepam
- Fluoxetine
- Hydrocodone
- Hydromorphone
- Hydroxyzine
- Ketamine
- Methamphetamine/MDMA
- Meperidine
- Methadone
- Methadone/LAAM
- Opiates
- Oxycodone/Oxymorphone
- PCP
- Promazine Group
- Propoxyphene
- Sertraline
- THC
- Trazodone
- Tricyclics Group
- Zolpidem

### Stimulants

- Amphetamine
- Amphetamine Specific
- Benzylpiperazine (BZP)
- Caffeine
- Cocaine/BZE
- Cotinine
- Doxapram
- Mazindol/Mazindol Metabolites
- Methylphenidate
- Methamphetamine/MDMA
- Nikethamide

### Anabolic Steroids/Beta Agonists

#### Anabolic Steroids

- 16 $\beta$ -Stanozolol
- Boldenone
- Nandrolone
- Stanozolol
- Trenbolone

#### Beta Agonists

- Bronchodilators
- Clenbuterol
- Ractopamine
- Theophylline

### Hallucinogens & Anesthetics

- Dextromethorphan
- Ketamine
- Lidocaine
- LSD
- Mepivacaine
- PCP
- Procaine

### Antidepressants/Sedatives

- Azaperone
- Barbiturates
- Benzodiazepines (Oxazepam/Clonazepam)
- Buspirone
- Citalopram
- Detomidine
- Flunitrazepam
- Fluoxetine
- Hydroxyzine
- Reserpine
- Sertraline
- Trazodone
- Tricyclics Group
- Zolpidem

### Narcotic Analgesics

- Alfentanil
- Anileridine
- Buprenorphine
- Buprenorphine/Norbuprenorphine
- Butorphanol
- Carfentanil
- Diprenorphine
- Etorphine
- Fentanyl (Remifentanil)
- Hydromorphone
- Levallorphan
- Lofentanil
- Meperidine
- Methadone
- Methadone/LAAM
- Naltrexone/Nalbuphine
- Opiates
- Oxycodone/Oxymorphone
- Pentazocine
- Propoxyphene
- Sufentanil
- Tramadol

### Anti-Inflammatories

- Acetaminophen
- Carprofen
- Celecoxib
- Corticosteroid Group
- Dexamethasone
- Flunixin
- Ibuprofen
- Ketorolac
- Methylprednisolone
- Phenylbutazone
- Triamcinolone Acetonide
- Zomepirac

### Antipsychotics

- Droperidol
- Fluphenazine
- Haloperidol
- Haloperidol Metabolites
- Promazine Group
- Tricyclics Group

### Pain Management & Prescriptions

- Amphetamine Specific
- Amphetamine Ultra
- Barbiturate Group
- Benzodiazepines (Oxazepam/Clonazepam)
- Buprenorphine
- Buprenorphine/Norbuprenorphine
- Butorphanol
- Carisoprodol
- Fentanyl
- Hydrocodone
- Hydrochlorothiazide
- Hydromorphone
- Meperidine
- Methadone
- Methadone/LAAM
- Methamphetamine/MDMA
- Methylphenidate
- Naltrexone/Nalbuphine
- Opiate Group
- Oxycodone/Oxymorphone
- Phenytoin
- Propoxyphene
- Tramadol
- Trazodone
- Zolpidem



# Neogen ELISA Drug Detection

## Cross-Reactivity Data

### Acetaminophen #132419

Acetaminophen	100%	Penicillin G-Procaïne	0.3%
Procainamide	2.1%	Procaine	0.1%
Ethyl p-aminobenzoate	1%	Methylene Blue	0.09%
Oxyphenbutazone	0.7%		

### Amphetamine #105219-1

d-Methamphetamine	1000%	α-Ethyltryptamine	1.2%
N-desmethylselegiline	203%	Phendimetrazine	0.7%
d-Amphetamine	100%	4-Hydroxyamphetamine	0.6%
Fenfluramine	87%	p-Hydroxymethamphetamine	0.5%
Ephedrine	56%	phenamine	0.5%
Nylidrin	21%	Tuaminoheptane	0.5%
l-Amphetamine	17%	MDA (3,4-methylenedioxyamphetamine)	0.4%
Mephentermine	14%	oxyamphetamine	0.4%
Phentermine	11%	Phenylephrine	0.4%
Phenethylamine	9.7%	Amitriptyline	0.06%
Benzphetamine	8.3%	Quinidine	0.06%
Phenylpropanolamine	6.3%	Tyramine	0.03%
Isoxsuprine	5.6%	Imipramine	0.02%
MDMA (3,4-methylenedioxyamphetamine)	2.2%	Glutethimide	0.01%
Pseudoephedrine	2.2%	Nortriptyline	0.01%
Clenbuterol	1.6%	Diethylpropion	<0.5%
Fencamfamine	1.6%	Hordeanine	<0.1%
		Pyrantel	<0.1%

### Amphetamine Specific #132319

p-Hydroxyamphetamine	162%	Phenethylamine	0.3%
p-Methoxyamphetamine	160%	2-Aminoheptane	0.2%
(+/-)-MDA	117%	d-Methamphetamine	0.1%
d-Amphetamine	100%	N-Desmethylselegiline	0.1%
α-Ethyltryptamine	5.3%	Mephentermine	0.06%
Phentermine	1.3%	p-Methoxy-methamphetamine	0.03%
Phenylpropanolamine	0.9%	(+/-)-MDMA	0.01%
Metaraminol	0.4%	S(-)-Methcathinone	0.01%
S(-)-Cathinone	0.4%		
l-Amphetamine	0.3%		

### Amphetamine Ultra #130819

N-desmethylselegiline	906%	2-Aminoheptane	0.6%
d-Methamphetamine	688%	Phenethylamine	0.5%
d-Amphetamine	100%	p-Hydroxyamphetamine	0.5%
(-)-Ephedrine	49%	Selegiline	0.4%
Fenfluramine	33%	(+/-)-MDA	0.4%
Mephentermine	25%	Phendimetrazine	0.4%
Nylidrin	14%	Clenbuterol	0.3%
l-Amphetamine	7%	Metaraminol	0.2%
Phenylpropanolamine	6%	p-Methoxymethamphetamine	0.2%
Phentermine	6%	phenamine	0.2%
Isoxsuprine	4%	p-Methoxyamphetamine	0.1%
Benzphetamine	2%	Hordeanine	0.03%
R(+)-Methcathinone	1%	Pyrimethamine	0.03%
(+/-)-MDMA	1%	Aminorex	0.03%
(+/-)-Pseudoephedrine	1%	Nicotine	0.02%
Fencamfamine	1%	Amitriptyline	0.02%
α-Ethyltryptamine	0.8%	Quinidine	0.02%
S(-)-Methcathinone	0.7%	Imipramine	0.01%
S(-)-Cathinone	0.6%	Nortriptyline	0.01%

### Barbiturate Group #130619

Secobarbital	100%	Butalbital	109%
Amobarbital	757%	Aprobarbital	98%
Phenobarbital	469%	Pentobarbital	95%
p-OH Phenobarbital	316%	Mephobarbital	0.3%
Barbital	239%	Methylene Blue	0.1%
Butobarbital	131%	Glutethamide	0.1%
Allobarbital	117%		

### Benzodiazepine Group (Oxazepam/Clonazepam) #130119

Oxazepam	100%	Nitrazepam	141%
Diazepam	434%	N-desmethyl	
Estazolam	365%	Flunitrazepam	119%
Nordiazepam	361%	Flunitrazepam	110%
Alprazolam	346%	Bromazepam	85%
Flurazepam	262%	Clonazepam	79%
Lormetazepam	231%	Lorazepam	70%
Prazepam	198%	Midazolam	65%
Temazepam	192%	Clobazam	59%
Halazepam	173%	7-amino-Clonazepam	39%
Triazolam	171%	alpha-hydroxy Alprazolam	25%
7-amino-Flunitrazepam	147%	Chlordiazepoxide	19%

### Benzylpiperazine #108319

Benzylpiperazine	100%	Isoxuprine	0.05%
1-(3-Trifluoromethyl)benzyl) piperazine	3.0%	N-[3-(Trifluoromethyl)phenyl] piperazine	0.03%
1-(4-Trifluoromethyl)benzyl) piperazine	1.0%	Imipramine	0.02%
Trazodone	1.0%	Acepromazine	0.02%
Nefazodone	0.20%	Orphenadrine	0.01%
Doxepin	0.13%	Amitriptyline	0.01%
Chlorpromazine	0.07%	Nicotine	0.01%
m-CPP	0.07%		

### Buprenorphine #131919

Diprenorphine	110%	Butorphanol	0.16%
Buprenorphine	100%	Naltrexone	0.05%
Norbuprenorphine	15%	Thebaine	0.02%
Etorphine	0.22%	Nalorphine	0.02%

### Buprenorphine/Norbuprenorphine #BN-96

Concentration (ng/mL)*			
Buprenorphine-glucuronide	4.5	Norbuprenorphine-glucuronide	10.2
Norbuprenorphine	5.0		
Buprenorphine	8.5		

### Carisoprodol #132519

Carisoprodol	100%
--------------	------

### Celecoxib #180719

Celecoxib Carboxylic Acid	143%	Celecoxib	100%
---------------------------	------	-----------	------

### Citalopram #132719

N-desmethylcitalopram	109%	Doxepin	0.03%
Citalopram	100%	Trimipramine	0.03%
(S)-Citalopram	20%	Nortriptyline	0.02%
Acepromazine	2%	Fluvoxamine	0.02%
Chlorpromazine	0.7%	Promazine	0.02%
Fluoxetine	0.1%	Orphenadrine	0.01%
Imipramine	0.07%	Sertraline	0.01%
Amitriptyline	0.03%	Duloxetine	0.01%
Paroxetine	0.03%		

### Cocaine/Benzoylecgonine #130319

Cocaine	133%	Ecgonine	0.4%
Cocaethylene	124%	Norcocaine	0.3%
Benzoylecgonine	100%	Ecgonine Methyl Ester	0.06%
m-Hydroxycocaine	96%		

### Cotinine #CT-96

Concentration (ng/mL)*			
(-) Cotinine	25	(R/S) Norcotinine	1750
(+/-) Trans-3'-hydroxycotinine	60		

### Detomidine #181319

Detomidine	100%	Medetomidine	26%
Carboxydetomidine	74%		

### Dextromethorphan #131419

3-Methoxymorphinan	118%	Orphenadrine	0.04%
Dextromethorphan	100%	Imipramine	0.02%
3-Hydroxymorphinan	1.7%	Nortriptyline	0.02%
Dextrophan	1.6%	Amitriptyline	0.02%
Promazine	0.05%	Acepromazine	0.01%
Trimipramine	0.04%	Meperidine	0.01%

### Fentanyl Group #100519

Sufentanil	270%	Thienylfentanyl	1.1%
Norsufentanil	119%	α-Methylfentanyl	0.77%
Alfentanil	100%	p-Fluorofentanyl	0.19%
Carfentanil	88%	3-Methylfentanyl	0.08%
Remifentanil	76%	Norfentanyl	0.04%
Fentanyl	2.0%		

### Fentanyl #131519

p-Fluorofentanyl	136%	Carfentanil	6%
Fentanyl	100%	Despropionylfentanyl	0.61%
Thienylfentanyl	67%	Lofentanil	0.20%
3-Methylfentanyl	50%	Sufentanil	0.15%
α-Methylfentanyl	11%	Norfentanyl	0.01%

### Fexofenadine/Terfenadine #181419

Fexofenadine	100%	Imipramine	0.03%
Terfenadine	23%	Nortriptyline	0.03%
Orphenadrine	0.20%	Promazine	0.03%
Hydroxyebastine	0.13%	Chlorpromazine	0.02%
Carebastine	0.07%	Ebastine	0.02%
Doxepin	0.04%	Trimipramine	0.01%
Amitriptyline	0.03%		

### Flunitrazepam #109519

7-Aminoflunitrazepam	212%	Halazepam	0.4%
Flunitrazepam	100%	Triazolam	0.3%
Diazepam	69%	Nitrazepam	0.3%
Clobazam	3.3%	Nordiazepam	0.2%
Clonazepam	2.1%	Methylene Blue	0.02%
Midazolam Maleate	1.4%	Trimipramine	0.01%
7-Aminoclonazepam	0.6%		
N-Desmethylflunitrazepam	0.5%		

### Fluoxetine #107619

Fluoxetine	100%	Norfluoxetine	67%
------------	------	---------------	-----

### Haloperidol #104319-1

Haloperidol	100%	Siperone	0.04%
Bromperidol	83%	Fluanisone	0.03%
Trifluoperidol	0.39%	Benperidol	0.01%
Risperidone	0.12%		

\* Concentration resulting in 50% drug inhibition

With over 20 years of developing and manufacturing ELISA test kits, Neogen's immunoassays are high quality and dependable with excellent reproducibility and sensitivity.

Our product line of over 100 immunoassays can screen 300+ drugs and metabolites in forensic matrices.

Test kits can be run manually or easily automated on a wide range of immunoassay instruments.

### Haloperidol Metabolites #102119

Bromperidol	157%
Haloperidol Metabolite (I) ((+)-4-(4-chlorophenyl)- $\alpha$ -(4-fluorophenyl)-4-hydroxy-1-piperidinebutanol)	100%
Haloperidol	84%
Haloperidol Metabolite (I) (4-(4-chlorophenyl)-4-hydroxypiperidine)	48%
Azaperone	0.002%

### Hydrochlorothiazide #180319

Trichlormethiazide	206%	Furosemide	0.24%
Hydrochlorothiazide	100%	Methylene Blue	0.01%
Chlorothiazide	8%		

### Hydromorphone #132219

6-Acetylcodeine	143%	Thebaine	2%
Hydromorphone	100%	Levallorphan	0.5%
Morphine	85%	Morphine-3-glucuronide	0.4%
Hydrocodone	72%	Oxycodone	0.3%
Dihydromorphone	56%	Oxycodone	0.2%
Codeine	55%	Nalorphine	0.1%
Ethylmorphine	54%	Normorphine	0.1%
Levorphanol	49%	Norcodeine	0.04%
6-Acetylmorphine	44%	Meperidine	0.02%
Morphine-6-glucuronide	39%	Dextromethorphan	0.02%
Dihydrocodeine	38%	Amitriptyline	0.01%
Heroin	29%	Doxepin	0.01%

### Ibuprofen #180219

Ibuprofen	100%	Indoprofen	0.02%
Flurbiprofen	0.3%	Suprofen	0.01%
Fenoprofen	0.06%		

### Ketamine #131719

Ketamine	100%	Norketamine	5.1%
----------	------	-------------	------

### LSD #130219

LSD	100%	2-oxo-3-OH LAMPA	1.2%
LAMPA	153%	Clenbuterol	0.05%
Nor-/Nor-iso-LSD	36%	Methylene Blue	0.04%
Iso-LSD	2.6%	Procaineamide	0.02%
2-oxo-3-OH LSD	2.1%		

### Meperidine #133119

Meperidine	100%	Diphenoxylate	0.06%
Normeperidine	2.3%	Chlorpromazine	0.01%
Anileridine	0.12%		

### Methadone #131619

LAAM	113%	Orphenadrine	0.01%
Methadone	100%	Acepromazine	0.01%
Propiomazine	0.07%	Amitriptyline	0.01%
Trimepazine	0.04%	Prochlorperazine	0.01%
Promethazine	0.04%	Promazine	0.01%
Trimipramine	0.02%	Chlorpromazine	0.01%
Imipramine	0.01%	Methotrimeprazine	0.01%

### Methadone/LAAM #132919

Methadone	100%	Acepromazine	0.03%
LAAM	107%	Chlorpromazine	0.03%
Dextromethorphan	0.18%	Imipramine	0.02%
Promethazine	0.15%	Doxepin	0.02%
Trimepazine	0.11%	Orphenadrine	0.02%
Trimipramine	0.09%	Amitriptyline	0.02%
EDDP	0.04%	Prochlorperazine	0.01%
Promazine	0.04%	Methotrimeprazine	0.01%
Propiomazine	0.03%	Meperidine	0.01%

### Methamphetamine/MDMA #130919

MDMA (3,4-Methylenedioxyamphetamines)	321%	Pseudoephedrine	0.8%
<i>d</i> -Methamphetamine	100%	<i>d</i> -Amphetamine	0.8%
<i>p</i> -Hydroxymethamphetamine	96%	4-Hydroxyamphetamine	0.5%
<i>l</i> -Methamphetamine	10%	N-desmethylephedrine	0.5%
Ephedrine	3%	$\alpha$ -Ethyltryptamine	0.4%
Mephentermine	3%	Phenethylamine	0.4%
Fenfluramine	2%	Phentermine	0.2%
MDA (3,4-Methylenedioxyamphetamine)	2%	Orderine	0.06%
		Nortriptyline	0.03%
		Procaine	0.01%

### Methylphenidate #133219

Methylphenidate	100%
Ritalinic Acid	0.53%

### Naltrexone/Nalbuphine #133019

Nalbuphine	202%	3-Methoxynaltrexone	6.3%
Naltrexone	100%	Naloxone	4.0%
Naltriben	41%	Nalorphine	1.3%
Butorphanol	28%	Pentazocine	0.3%
Nalmefene	26%	Codeine	0.02%
6 $\beta$ -Naltrexol	6.5%		

### Opiate Group #130419

6-Acetylcodeine	195%	Naloxone	2%
Codeine	190%	Oxycodone	2%
Morphine-3-Glucuronide	154%	Levorphanol	2%
Heroin/Diacetylmorphine	154%	Oxymorphone	1%
6-Acetylmorphine	146%	Imipramine	0.4%
Hydrocodone	122%	Norcodeine	0.3%
Ethylmorphine	110%	Trimipramine	0.3%
Morphine	100%	Normorphine	0.2%
Nalorphine	76%	Naltrexone	0.2%
Hydromorphone	66%	Acepromazine	0.05%
Morphine-6-Glucuronide	37%	Meperidine	0.02%
Thebaine	11%	Chlorpromazine	0.02%
Levallorphan	7%	Promazine	0.01%

### Oxycodone/Oxymorphone #130719

Oxycodone	100%	6-Acetylcodeine	3%
Oxymorphone	88%	6-Acetylmorphine	2%
Hydrocodone	27%	Levorphanol	2%
Hydromorphone	21%	Heroin	2%
Ethylmorphine	7%	Morphine	1%
Codeine	5%	Noroxymorphone	0.2%
Thebaine	3%	Meperidine	0.03%

### Pentazocine #103119

Pentazocine	100%	Carfentanil	0.01%
-------------	------	-------------	-------

### Phencyclidine (PCP) #130519

Phencyclidine	100%	Nortriptyline	0.02%
4-Hydroxyphencyclidine	1.8%	Chlorpromazine	0.02%
Venlafaxine	0.10%	Imipramine	0.02%
Cyclobenzaprine	0.05%	Doxepin	0.01%
Orphenadrine	0.04%	Methylene Blue	0.01%
Amitriptyline	0.03%	Trimipramine	0.01%
Promazine	0.03%	Acepromazine	0.01%

### Phenytol #132119

Phenytol	100%	Hydroxyphenytol	61%
----------	------	-----------------	-----

### Propoxyphene #131119

Propoxyphene	100%	Norpropoxyphene	30%
--------------	------	-----------------	-----

### Sertraline #131319

Sertraline	100%	Amitriptyline	0.2%
N-Desmethylsertraline	50%	Acepromazine	0.2%
Methylene Blue	4.3%	Nortriptyline	0.2%
Fluoxetine	0.7%	Doxepin	0.1%
Pyrimidine	0.4%	Pyrethamine	0.1%
Paroxetine	0.3%	Trimipramine	0.1%
Promazine	0.2%	Imipramine	0.1%
Chlorpromazine	0.2%	Phenothiazine	0.02%

### THC #131019

$\Delta^9$ -THC-COOH	100%	$\Delta^9$ -THC	4%
$\Delta^8$ -THC-COOH	88%	Cannabinol	2%
11-OH- $\Delta^9$ -THC	13%	Cannabidiol	0.01%
$\Delta^8$ -THC	5%		

### Tramadol #131819

Tramadol	100%	N,N-Bisdesmethyl	
O-Desmethyl Tramadol	22%	Tramadol	4.7%
N-Desmethyl Tramadol	6.7%		

### Trazodone #132819

Trazodone	100%	Doxepin	0.1%
Nefazodone	15%	1-(4-(Trifluoromethyl)benzyl)piperazine	0.07%
m-CPP	0.5%	Acepromazine	0.01%
N-(3-(Trifluoromethyl)phenyl)piperazine	0.2%	Imipramine	0.01%
1-(3-(Trifluoromethyl)benzyl)piperazine	0.2%		

### Tricyclics Group #132019

Amitriptyline	464%	Trifluoperazine	2.6%
Doxepin	364%	Thioridazine	1.7%
Cyclobenzaprine	315%	Orphenadrine	1.5%
Dothiepin	232%	Acetophenazine	0.7%
Nortriptyline	155%	Acepromazine	0.6%
Clomipramine	143%	Clozapine	0.4%
Imipramine	100%	Thiethylperazine	0.2%
Protriptyline	44%	Propionylpromazine	0.1%
Trimipramine	44%	Mesoridazine	0.04%
Promazine	21%	Thiothixene	0.04%
Chlorpromazine	20%	Glycopyrrolate	0.02%
Desipramine	12%	Olanzapine	0.02%
Perphenazine	8.7%	Nefopam	0.02%
Prochlorperazine	8.4%	Hydroxyzine	0.02%
Cyproheptadine	4.2%	Trazodone	0.01%
Maprotiline	3.9%	Pyrimidine	0.01%
Trifluoperazine	3.4%	Pyrimethamine	0.01%
Fluphenazine	3.0%		

### Zolpidem #132619

Zolpidem	100%	Carboxy-Zolpidem	0.02%
----------	------	------------------	-------



# Neogen ELISA Drug Detection

## Forensic Products

ELISA Test Kit	Product #	ELISA Test Kit	Product #	ELISA Test Kit	Product #
Acetaminophen	132419	Fentanyl	131519	Phencyclidine (PCP)	130519
Alfentanil	103619	Fexofenadine/Terfenadine <b>NEW</b>	181419	Phenylbutazone	104719-1
Amphetamine	105219-1	Flunitrazepam	109519	Phenytoin	132119
Amphetamine Specific	132319	Flunixin	101919	Procaine	103219
Amphetamine Ultra	130819	Fluoxetine	107619	Promazine Group	100719
Anileridine	105519	Fluphenazine	104119	Propoxyphene	131119
Apomorphine/Apocodeine	109119	Furosemide	104219-1	Propranolol	107319
Azaperone	100919	Glycopyrrolate	102019	Pyrilamine	105919
Barbiturate Group	130619	Guanabenz	109219	Ractopamine	106519
Benzodiazepine (Oxazepam/Clonazepam)	130119	Haloperidol	104319-1	Reserpine	104819
Benzylpiperazine	108319	Haloperidol Metabolites	102119	Sertraline	131319
Boldenone	101019	Hydrochlorothiazide	180319	Stanozolol	103319
Bronchodilator Group	100319	Hydromorphone	132219	16 $\beta$ -Hydroxystanozolol	103519
Bumetanide	103719-1	Hydroxyzine	105719	Sufentanil	104919
Buprenorphine	131919	Ibuprofen	180219	Sulfamethazine	103419
Buprenorphine/ Norbuprenorphine	BN-96	Ipratropium/Atropine	107119	THC	131019
Buspirone	108719	Isoxsuprine	102219	Theophylline	106019
Butorphanol	101119	Ketamine	131719	Tramadol	131819
Caffeine/Pentoxifylline	106419	Ketoprofen	108219	Trazodone	132819
Carfentanil	103919	Ketorolac	105619	Trenbolone	109719
Carisoprodol	132519	Levallorphan	102319	Triamcinolone Acetonide	105119
Carprofen	181119	Lidocaine	106719-1	Tricyclics Group	132019
Celecoxib	180719	Lofentanil	102419	Zolpidem	132619
Citalopram	132719	LSD	130219	Zomepirac	109619
Clenbuterol	101219	Mazindol/Mazindol Metabolites	102519		
Clonidine/Romifidine	180119	Meperidine <b>NEW</b>	133119		
Cocaine/Benzoylecgonine	130319	Mephentermine	107219		
Corticosteroid Group	100419	Mepivacaine	102719		
Cotinine	CT-96	Metaraminol	107919		
Cromoglycate	105819	Methadone	131619		
Dantrolene	106319	Methadone/LAAM <b>NEW</b>	132919		
Detomidine <b>NEW</b>	181319	Methamphetamine/MDMA	130919		
Dexamethasone	101519	Methocarbamol	108019		
Dextromethorphan	131419	Methotrexate	107519		
Diprenorphine	106119	Methylphenidate <b>NEW</b>	133219		
Doxapram	106219	Methylprednisolone	104519		
Droperidol	101619	Naltrexone/Nalbuphine <b>NEW</b>	133019		
Ethacrynic Acid	101719-1	Nandrolone	104619		
Etorphine	101819	Nikethamide	109919		
Fenspiride	108419	Opiate Group	130419		
Fentanil Group	100519	Oxycodone/Oxymorphone	130719		
		Pentazocine	103119		

### Hair Kits - **NEW**

Cocaine	140115
Methamphetamine/MDMA	140215
Opiates	140315
Synthetic Opiates	140415
PCP	140515
THC	140615