

IGENITY HAIR SAMPLE INSTRUCTIONS



PLEASE READ KIT INSTRUCTIONS COMPLETELY BEFORE PERFORMING TEST.

Call 877.443.6489 to order or visit NEOGEN.com



1 Pull (do not cut) hair from the tail switch, not tail head. Pull hair in the opposite direction from which the hair is laying. This results in less breakage than pulling straight out from the tail.



2 Make sure hair is free of fecal material and dirt. Save the hair roots (follicles), which contain the DNA.



3 Roots must be clearly visible. Approximately 30 hair roots are needed. For animals with finer hair, a minimum of 50 to 60 hairs is desired. For animals younger than three months, TSUs are recommended since hair roots will likely not be visible.



4 Open the collector and peel back the plastic cover. Insert the root end of hairs in the middle of the collector. Seal the plastic cover over the hair roots and then close the collector cover tab.



5 Trim excess hair extending from the collector. Write animal ID and other information in the spaces provided (do not cut off follicles, which contain the DNA).



6 Properly record animal IDs on the collector cards and correctly enter that information on your order form to match the DNA to the animal.

