

Polysulfated Glycosaminoglycan,  
Sterile Solution



# Chondroprotec<sup>®</sup>

Product Number: 08820



## Presentation $\mathcal{R}_x$

Chondroprotec	Product Number	Case Count
1,000 mg/10 mL vial	08820	3/box

## Indications

Chondroprotec<sup>®</sup> is a sterile 10 mL solution containing 1,000 mg of Polysulfated Glycosaminoglycan (100 mg/mL).

For use on dogs, cats, and horses. Used to treat partial and full thickness wounds, secreting and bleeding dermal lesions, trauma injuries, first and second degree burns, pressure ulcers (stages I-IV), surgical incisions, donor sites, and venous stasis ulcers.

## Dosage and Administration

Irrigate wound with normal saline solution.

Either pour contents of vial or dispense from syringe to withdraw solution and cover wound site.

Cover with a non-adherent dressing such as PUF (polyurethane film) or Telfa<sup>®</sup>. Note: Subsequent applications of Chondroprotec can be injected through the dressing as to not disturb the wound bed.

Reapply Chondroprotec daily and redress wound site every 5-7 days or as needed.

## Features

- 1,000 mg of PSGAG in a 10 mL via (1,000 mg/mL)
- Derived from bovine trachea
- Preservative-free
- Patented technology

## Caution

No known side effects; not intended for use as long term or permanent dressing. Federal law restricts this device to sale by or on the order of a licensed veterinarian.

## Warning

Not for food animal use. Not for human use.

## Storage

Store at a temperature between 15°-30° C (56°-86° F). Protect from freezing. Sterile when unopened. Discard unused portion.

To learn more, call 800.525.2022 or 859.254.1221, or visit [neogen.com](http://neogen.com).

