

# CHONDROPROTEC®

## Polysulfated Glycosaminoglycan

NEOGEN Vet

Net Contents: 10 mL/1000 mg

### INDICATIONS FOR USE:

- Donor Sites
- Trauma Injuries
- Surgical Incisions
- Venous Stasis Ulcers
- 1st and 2nd Degree Burns
- Pressure Ulcers (Stages I-IV)
- Partial and Full Thickness Wounds
- Secreting and Bleeding Dermal Lesions

### FOR VETERINARY USE ONLY



Distributed by: Neogen Corporation  
Lexington, KY 40511 • 800.525.2022  
NEOGEN.com L145-1220

### DIRECTIONS FOR USE:

1. Irrigate wound with normal saline solution.
2. Either pour contents of vial or dispense from syringe to withdraw solution and cover wound site.
3. Cover with a non-adherent dressing such as PUF (polyurethane film) or Telfa®. Note: Subsequent applications of CHONDROPROTEC can be injected through the dressing as to not disturb the wound bed.
4. Reapply CHONDROPROTEC daily and redress wound site every 5 - 7 days or as needed.

Store at a temperature between 15° and 30°C (59°F - 86°F).

Protect from freezing. Discard unused portion.

**Sterile when unopened.**

For use on dogs, cats, and horses.

Not for food animal use.

NO UV

Lot No.

Exp. Date