



Simplifying Oral Fluid Collection

NEOGEN® is proud to announce the release of our NeoSal® Oral Fluid Collection System. Our oral fluid collection system is designed for maximum ease of use at the collection site and testing laboratory.

Confident Collection

- Sample volume adequacy indicator: Ensures appropriate and consistent sample volume is collected.
- Stimulant-free collection pad: Collects sufficient volume without artificial stimulants.
- Quick collection: Typical collection time is 1–2 minutes.

Engineered Ease of Use

- Unique collector handle design: Handle remains outside of the buffer tube, reducing risk of sample contamination and forming a leak-proof seal for long term storage.
- No additional processing: No centrifugation or plunging necessary, discard collection pad and process sample.
- Automation friendly: Buffer tube fits onto most standard automated instrument sample racks (12 x 75), eliminating transfer to additional tube.
- Additional cap: Designed for long-term storage, stores in bottom of tube when not in use.

Optimized Performance

- Oral fluid ELISA kits: Optimized with the NeoSal, samples can be assayed without additional sample preparation.
- Drug recovery: Innovative proprietary buffer extracts adequate analyte concentration for consistent detection.
- Sample stability: Buffer contains microbial growth inhibitors and ensures stability of the sample during transport.
- Compatibility: NeoSal buffer is compatible with liquid chromatographic separation approaches and mass spectrometer detection methods.

Oral Fluid Collection System Ordering Information

Product	Available Sizes	Product No.
NeoSal Oral Fluid Collection System	25 pack	128101-25
NeoSal Buffer	500 mL 1 L	120176 120177
Buffer Tube caps	25 pack	128102-25


Call 800.477.8201 to order or visit NEOGEN.com

NEOGEN.com


Simplifying Oral Fluid Collection

Simple Collection Process


The NeoSal Oral Fluid Collection System is designed for maximum ease of use at the collection site and testing laboratory. Collection is completed with these steps:

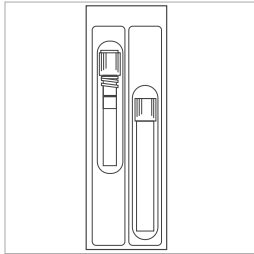


Peel open package and remove collector. Do not eat or drink at least 15 minutes prior to collection.

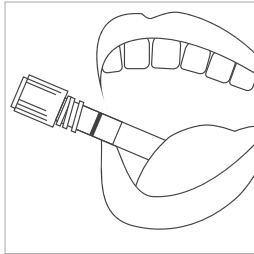


Pool saliva in mouth. Insert collector pad in mouth between cheek and gum.

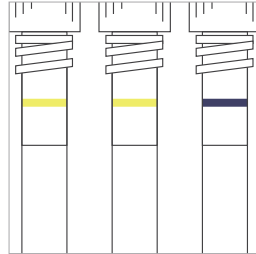

When a distinct blue line forms, remove collector from mouth. Typical collection time is 1–2 minutes.

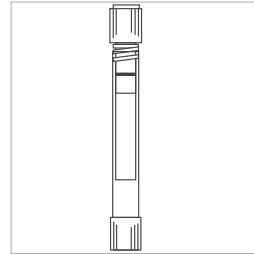

Remove blue cap and snap into bottom of the tube. Insert saturated collector pad into tube.

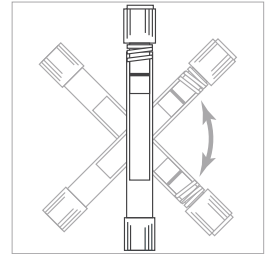

Tighten white cap. Gently invert tube 4 or 5 times. Send sample to testing laboratory.

NEOGEN's Drug Detection (Ready to Use) Forensic Assays

NEOGEN's forensic drug detection immunoassays include more than 100 ELISA test kits to screen over 300 drugs and metabolites in a wide range of forensic samples including whole blood, urine, oral fluid, and others.

ELISA Test Kits	Product No.	ELISA Test Kits	Product No.	Oral Fluid ELISA Kits	Product No.
6-AM	134019	Phencyclidine (PCP)	130519	Cocaine/BZE	120119
Acetaminophen	132419	Phenytoin	132119	Methamphetamine/MDMA	120219
Amphetamine Specific-2	132319-2	Propoxyphene	131119	Opiates	120319
Amphetamine Ultra	130819	Salicylates	133619	PCP	120419
Barbiturate Group	130619	Sertraline	131319	THC	120519
Benzodiazepine Group (oxazepam/clonazepam)	130119	Synthetic Cannabinoids (JWH-018)	133519	Oral Fluid Multi-Analyte Calibrator Set	128100
Buprenorphine	131919	Synthetic Cannabinoids (JWH-250)	133719	See additional kits at NEOGEN.com	
Carisoprodol	132519	Synthetic Cannabinoids (UR-144/XLR-11)	133919	FORENSIC USE ONLY	
Citalopram	132719	Synthetic Cathinones (Methcathinone)	181819		
Cocaine/Benzoyllecgonine-2	130319-2	THC	131019		
Dextromethorphan	131419	Tramadol	131819		
Fentanyl	131519	Trazodone	132819		
Hydromorphone	132219	Tricyclics Group	132019		
Ketamine	131719	Zolpidem	132619		
LSD	130219				
Meperidine	131219				
Methadone	131619				
Methadone/LAAM	132919				
Methamphetamine/MDMA	130919				
Methylphenidate/Ritalinic Acid	134219				
Naltrexone/Nalbuphine	133019				
Opiate Group	130419				
Oxycodone/Oxymorphone	130719				

Call 800.477.8201 to order or visit NEOGEN.com

